


Stamen Device

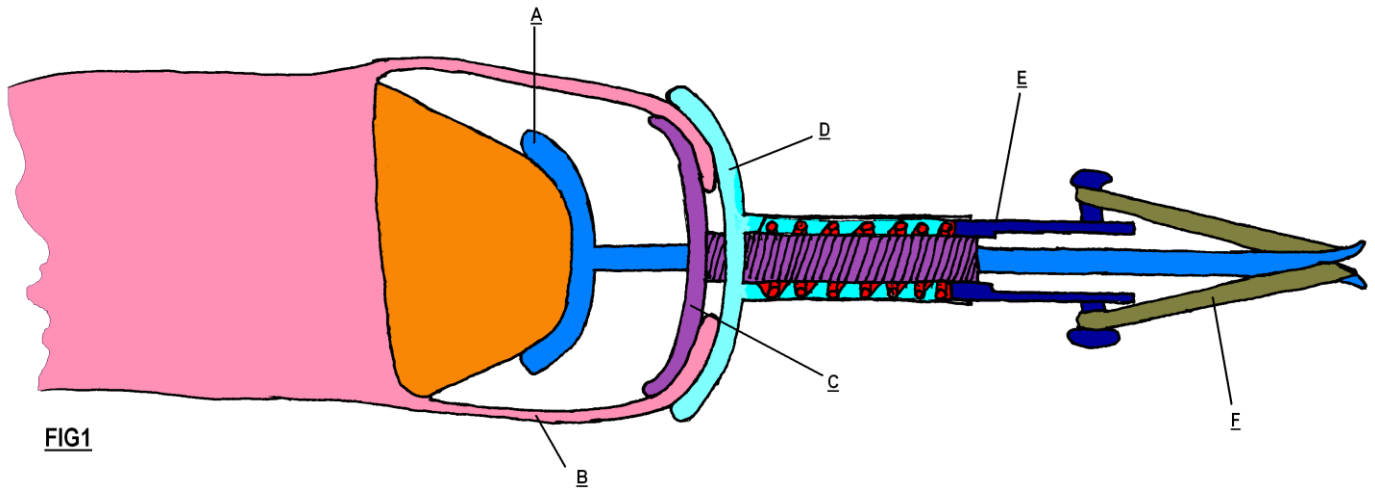

A Glans Base

C Base holding bell

E Spring adjuster

B Foreskin

D Top holding bell

Fitting Instructions

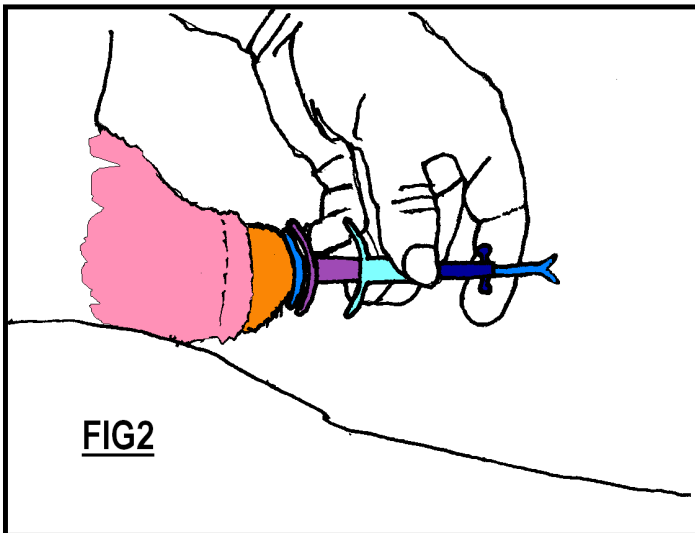


FIG2

Place Glans base A against Glans

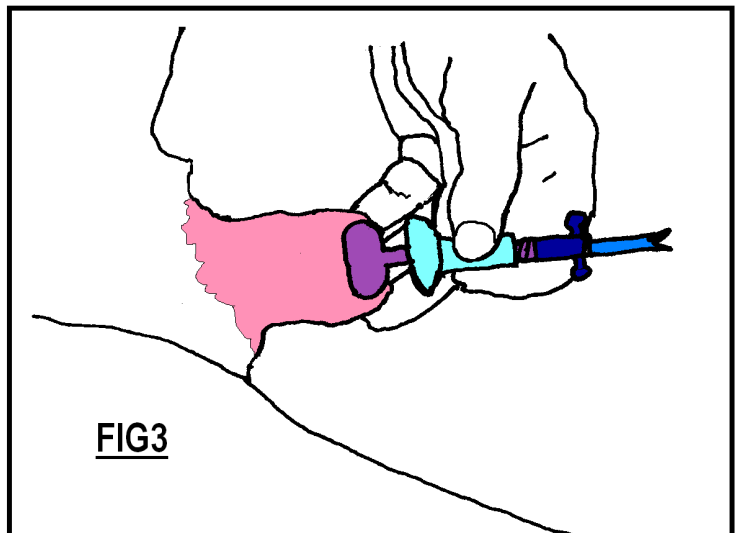


FIG3

Roll forward Shaft Skin B onto Base Holding Bell C

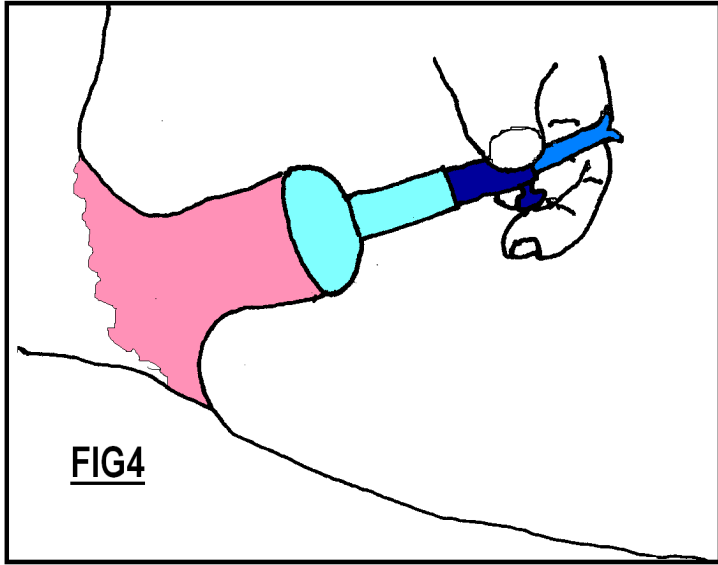


FIG4

And bring **D** Top Holding Bell onto shaft Skin **B**. Now gently adjust **E** by carefully turning until the pressure feels comfortable

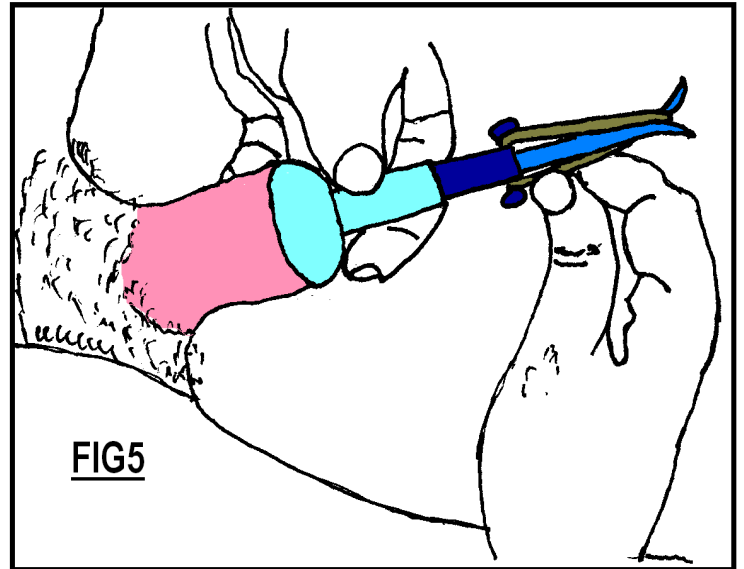


FIG5

Carefully apply a suitable elastic band. Be careful not to use too high a tension as one only requires quite a light pressure

Measurement

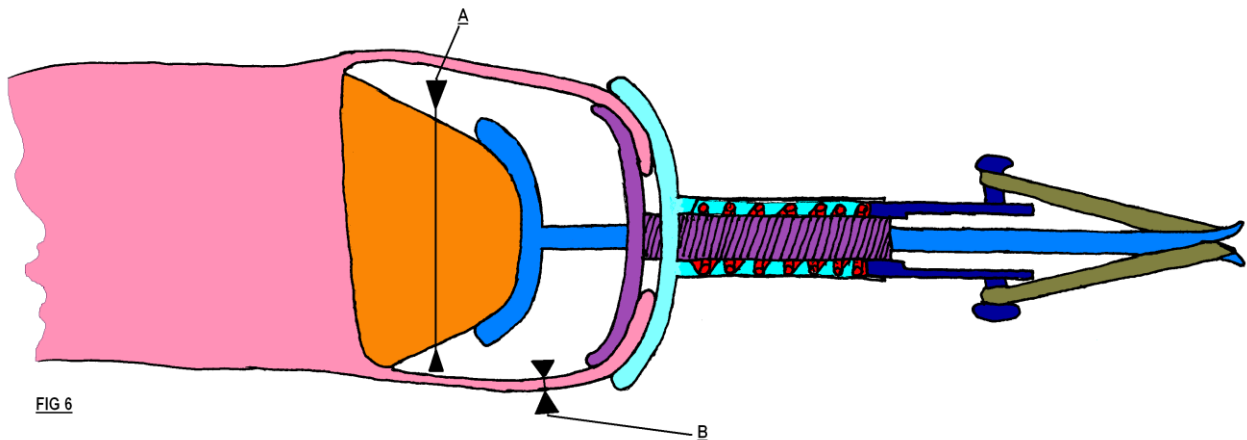


FIG6

Each Stamen device is individually made to give the best possible fit for this we require two dimensions. Firstly at point **A** we require the approximate diameter of the Glans and the second dimension at point **B**
Please supply us with these measurements.

Sheet 2

stemenproducts@yahoo.com

